






# FORA® Safety Lancet

ACS023

-  33G / 1.6mm
-  32G / 1.6mm
-  30G / 1.6mm
-  28G / 1.8mm
-  26G / 1.8mm



**Single Use:** No need for a lancing device, reduces trans-infection.

**Safe:** Needle retracts immediately after use which reduces the risk of unintentional injuries.

**Easy:** Twist off and press, no need to load lancing device and remove it.


**Fast:** One unit replaces two items which reduces time from loading and unloading the lancets.

**Accurate:** Controlled puncture depth reduces the pain and increases accuracy




**FORA® Safety Lancet (ACS023)** is designed to make blood sampling simple and easy. Touch activation means the lancet can be used in 3 easy steps.

**TWIST**

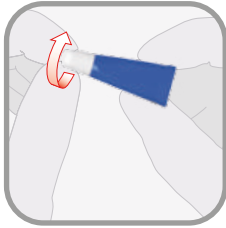

Twist off and discard lancet cap. **DO NOT PULL.**

**PRESS**

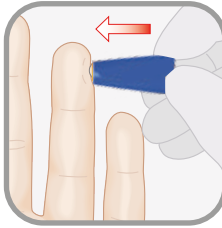


Press platform firmly against sample site. Lancet will automatically retract into body of device.

**DISPOSE**



Dispose of in a suitable sharps container.

Color	Gauge / Depth	Amount of bleeding
	33G / 1.6mm	
	32G / 1.6mm	
	30G / 1.6mm	
	28G / 1.8mm	
	26G / 1.8mm	

CE 0123 REF ACS023



**ForaCare Suisse AG**  
 Neugasse 55, 9000 St. Gallen, Switzerland  
 TEL : +41-71-220-1001 FAX : +41-71-220-1075  
 URL : www.foracare.ch  
 E-mail : chservice@foracare.com

